



KEEP YOUR DOG'S BODY COOL, WHEREVER YOU ARE!



## HOT WEATHER? HIKING? PLAYING? TRAVELLING?

With the DRY® Cooling Vest you can prevent dogs from overheating!

NOT LIMITED TO ONE SPOT COMPARED TO A COOLING MAT

READY WITHIN 1 MINUTE

COOLS DOWN UP TO 15°C / 59°F

PREMIUM QUALITY

COOLING EFFECTIVE ABOVE 20°C / 68°F

SAFEST COOLING SOLUTION FOR DOGS

USED BY THE ARMY

REFILLABLE UP TO 800 TIMES!

100% DRY

NO WET OR SMELLY FUR COMPARED TO A WET VEST

UP TO 3 DAYS UNBROKEN COOLING

RECOMMENDED BY VETERINARIANS WORLDWIDE

AVAILABLE SIZES:

XS

S

M

L

XL



Suitical

# DRY Cooling Vest



## HOW TO ACTIVATE THE DRY® COOLING VEST?


First open the valve cap and fill the vest with water. Check the chart below for the quantity of water required. Close the valve cap. Evenly distribute the water throughout the vest by gently pressing against it with your hands on all sides. Fold the vest, open the valve cap and gently squeeze out all excess water. Now close the valve cap. The water in the vest will immediately start the cooling process. No gels, refrigeration or other chemicals are needed. The vest will stay cool for up to 3 days. After using the vest, simply hang-dry for storage.

For a more detailed explanation, visit [www.suitical.com](http://www.suitical.com) for the video tutorial on how to activate the vest.




## PUTTING THE VEST ON YOUR DOG

Place your dog's head through the opening. Secure the vest by fastening the buckles from the belly to the back. Adjust the straps by pulling gently. Make sure the vest is not too tight around the chest and neck as this can restrict blood circulation.

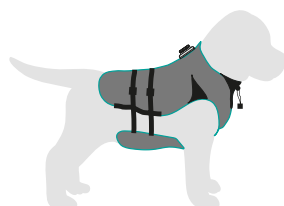


## PROFESSIONAL COOLING SYSTEM

1. Fill the vest with water;
2. Hot outside air evaporates the water;
3. The evaporation cools waterwick;
4. Surface of the vest cools down;
5. The cooling effect radiates through the membranes of the inner fabric on to your dog's body.

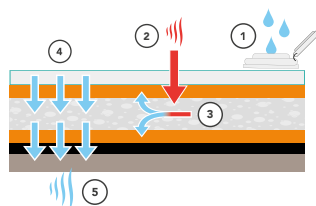

## MEASURE YOUR DOG CORRECTLY

For a comfortable fit, see the measurements below. If your dog is somewhere between two sizes, we recommend going for the larger size. When measuring the chest circumference, measure the widest part of your dog's chest. When measuring the neck, leave room for two fingers. When activating the vest, use the correct amount of water for your dog's size and gently squeeze out any excess water.

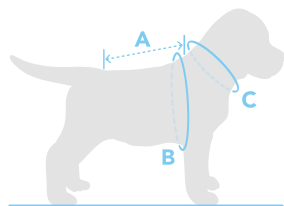

| SIZE   | LENGTH (A)<br>(base of neck to base of tail) | CHEST (B)<br>CIRCUMFERENCE | NECK (C)<br>CIRCUMFERENCE | WATER<br>REQUIRED  |
|--------|--|----------------------------|---------------------------|--------------------|
| XSMALL | 25 - 35 cm / 10 - 14 inch                    | 40 - 60 cm / 16 - 24 inch  | < 35 cm / < 14 inch       | 100 ml / 3.40 oz.  |
| SMALL  | 35 - 50 cm / 14 - 20 inch                    | 55 - 70 cm / 22 - 28 inch  | < 45 cm / < 18 inch       | 200 ml / 6.75 oz.  |
| MEDIUM | 50 - 60 cm / 20 - 24 inch                    | 60 - 80 cm / 24 - 32 inch  | < 50 cm / < 20 inch       | 290 ml / 10.20 oz. |
| LARGE  | 60 - 75 cm / 24 - 30 inch                    | 65 - 85 cm / 26 - 34 inch  | < 55 cm / < 22 inch       | 430 ml / 14.50 oz. |
| XLARGE | 70 - 90 cm / 28 - 36 inch                    | 75 - 95 cm / 30 - 38 inch  | < 65 cm / < 26 inch       | 525 ml / 17.70 oz. |

